AeroCarbon™ Rollator

Déambulateur AeroCarbon™

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10278GY

10278BK

www.drivemedical.com

parts list

WEIGHT CAPACITY: 330 LBS (150 KG)

- | | |
|------------------------------|-------------------|
| 1. Reflector (Seat Supports) | 9. Storage Bag |
| 2. Hand Grip | 10. Rear Caster |
| 3. Hand Brake Lever | 11. Curb Assist |
| 4. Height Adjustment Button | 12. Brake Pad |
| 5. Backrest | 13. Bag Mount |
| 6. Backrest Adjustment Knob | 14. Folding Latch |
| 7. Seat | 15. Cane Holder |
| 8. Seat Handle | |

AeroCarbon™ Rollator includes:

- | | |
|--------------------------|---------------------------|
| 1. Removable Storage Bag | 3. Cane Holder |
| 2. Adjustable Backrest | 4. Operating Instructions |

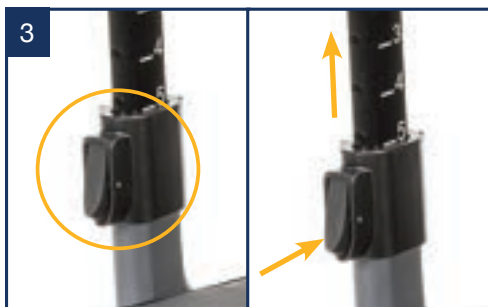
unfolding the AeroCarbon™ Rollator

To unfold, position the rollator on all four wheels and ensure all four wheels rest firmly on the ground. Undo the folding latch. Hold each hand grip and pull outward until the rollator unlocks and opens fully (figure 1). Gently push down on each side of the seat rails near the vertical frame to ensure the unit is fully opened (figure 2). An audible 'click' confirms that the latches are secure, and the seat is fully engaged in a locked position.

height adjustment

Press the push button for adjusting the height of the push handles and then pull these upwards at the same time (figure 3). An audible 'click' confirms the handle tube is secure in the locked position. Repeat this process until the desired height is reached for both handles. Refer to the numbering system (figure 4) to ensure both handles are adjusted to the same height.

backrest adjustment

Height Adjustment: Turn backrest adjustment knob counterclockwise to loosen. Slide backrest upward or downward to desired height then turn knob clockwise to tighten (figure 5).

DO NOT HANG ANYTHING ON THE BACKREST

removing / attaching the storage bag

To attach the storage bag to the rollator, affix the bag mounting loops (located on either side of the storage bag) to the seat rails on the rollator (figures 6-7).

WEIGHT CAPACITY OF THE BAG IS 11 LB.

DO NOT OVERLOAD AS IT WILL EFFECT PERFORMANCE.

braking / locking

Pull both brake levers upward to apply the brakes while the rollator is in motion (figure 8). Always brake slowly and steadily. While the rollator is stationary, push the brake lever down to engage the parking brake (figure 9). To disengage the locked brake, grip the brake lever and lift it upward until it audibly clicks into place, indicating it is now released. Releasing the locking brake may be easier with the use of both hands. Hold the handle with one hand while using the other hand to raise the brake lever.

DO NOT SQUEEZE BRAKE CONTINUOUSLY WHILE WALKING AS THIS WILL CAUSE PREMATURE WEAR TO THE TIRE AND BRAKE.

adjustment of brakes

Undo the locknut and rotate this fully upwards (Figure 10). At the same time fix the end sleeve of the cable. To adjust the braking force, rotate the end sleeve of the cable either clockwise or anticlockwise (Figure 11).

sitting

While facing the rollator, position the open and locked rollator on a flat, even, solid surface on all four wheels. Ensure the brakes are in the locked position. The rollator must remain upright and must not move. Once the brakes are securely engaged, utilize the handles for support, to turn completely around. With both hands on the handles slowly sit, ensuring you sit straight up and do not lean backwards (figure 12).

DO NOT MOVE ROLLATOR WHILE SEATED.

DO NOT USE THE ROLLATOR AS A WHEELCHAIR/TRANSPORT CHAIR. THE ROLLATOR IS NOT DESIGNED FOR THIS PURPOSE.

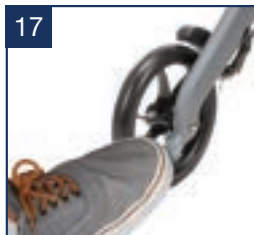
folding / transport / storage

To fold, pull the seat upward by the seat handle (figure 13). The rollator can now be folded completely and secured by pushing the hand grips together (figure 14) into the closed/folded position (figure 15).

curb assist

The AeroCarbon™ Rollator features a curb assist for terrain transitions. The curb assist allows the user to move over curbs and small steps with ease. The curb assist is visibly located on the inside of the rear wheels (figure 16 & 17).

HOW TO USE THE CURB ASSIST:

1. When approaching a curb, step or transition firmly grasp both hand grips and place your foot on the curb assist.
2. While securely grasping both hand grips, gently apply pressure to the curb assist with your foot.
3. Bring the front wheels of the rollator to a slight angled position just above the curb or step pushing forward if necessary.
4. Once in a stable position, gently release your foot from the curb assist to allow the front wheels to rest on the curb, step or transition.
5. Push the rollator forward as the rear wheels approach the curb, step or transition and lift the rear wheels onto the curb, step or transition.
6. Proceed with walking once you observe that all four wheels are safely past the curb, step or transition.

cleaning

Clean regularly by hand with a damp cloth and mild detergent only.

Do not use aggressive detergents such as chlorine, bleach or abrasive cleaners.

maintenance

Drive DeVilbiss Healthcare recommends periodic visual inspection of this product by the user to ensure that all parts and hardware are secure, and all components are in good working order and not worn, torn, frayed, or loose. There should be no obstructions or impediments to normal and safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience any problems or issues, please contact the provider that sold you the product or visit www.drivemedical.com/contact-us for more information.

safety instructions

- This is a walking aid only and is not to be used as a transportation device.
- Use only on even and solid surfaces.
- Maximum user weight: 330 lb.
- Regularly check the screws and fasteners on rollator and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 11 lb.
- Do not expose the product to temperatures above 104°F for an extended period as this can cause parts to overheat and burn the user.
- Please make sure the rollator is fully opened, seat is down and the folding bar under the seat is locked in the down position before using or sitting in the rollator.
- Be sure both hand grips are at equal heights. Test rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the rollator. When wheels are locked, the brake pad should prevent the wheels from turning. When the brakes are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tires when significantly worn or damaged.
- **DO NOT MOVE ROLLATOR WHILE SEATED.**

lifetime limited warranty

Your Drive® brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive® device or this warranty, please contact an authorized Drive® dealer.

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© 2025 Medical Depot, Inc. All rights reserved.
AeroCarbon™ is a registered trademark of Medical Depot, Inc.
© 2025 Medical Depot, Inc. Tous droits réservés.
AeroCarbon™ est une marque de commerce de Medical Depot, Inc.